

# Intersurgical Aquamist™

Humidifier Nebuliser




## Intersurgical Aquamist®

### Humidifier Nebuliser

The Intersurgical Aquamist® Humidifier Nebuliser has been designed to deliver accurate concentrations of humidified oxygen quietly. The Humidifier Nebuliser allows the clinician to prescribe a concentration of oxygen which will remain unaffected by the flow rate or the patient's own breathing pattern.

The Intersurgical Aquamist® Humidifier Nebuliser offers a range of accurate oxygen concentrations, which may be selected to suit the clinical requirements.

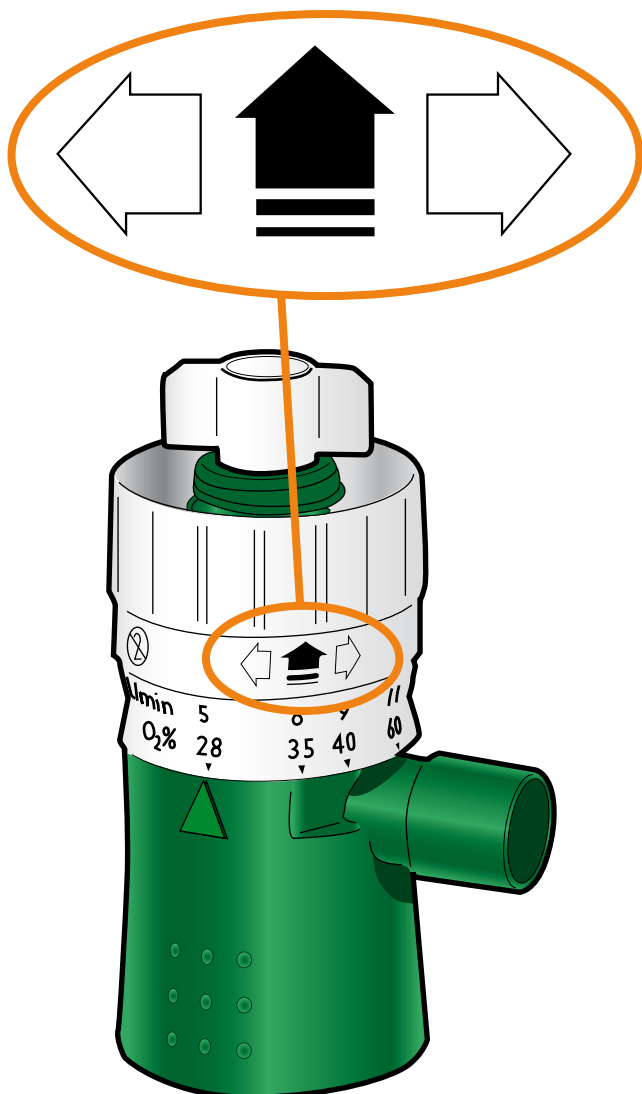

#### Quiet Operation

The Intersurgical Humidifier Nebuliser is exceptionally quiet in use – offering a noticeable difference compared to traditional products. This provides a more peaceful clinical environment.

#### High Humidification output

The combination of an accurate oxygen concentration and a high humidification output provides an ideal solution to humidification requirements.

#### Product Options

The Intersurgical Humidifier Nebuliser is compatible with the most popular standard, sterile water bottles including Baxter, Braun, and MPV Truma. It is also available with a refillable 500ml bottle if required.

It can be simply attached to the majority of flow meters via the universal screw thread fitting. The 1500 and 1502 options are supplied with a 1506-001 adaptor enabling them to be fitted to a BOC Flowmeter. An M12 adaptor, code 1509-001 is also available.

#### Tamper Resistant Oxygen Control Setting

The oxygen concentration selection is made by lifting and turning the top of the nebuliser and aligning the required oxygen percentage with the arrow as highlighted in the illustration left. This tamper resistant control prevents accidental concentration changes, providing a safer and more reliable product.

The Humidifier will operate with a driving gas flow of 5 to 11 l/min dependant on the oxygen concentration required

## Features and Benefits

### Low operating noise

Providing a quieter clinical environment

### Ease of setting

Recommended flow for each oxygen concentration

### High humidification output

Optimum level of humidification

### Tamper resistant oxygen control

Preventing accidental changes to oxygen control setting

### Accurate delivered oxygen concentrations


### Wide range of options available

Designed to meet your requirements




# Ordering information

## Aquamist™ humidifier nebulisers

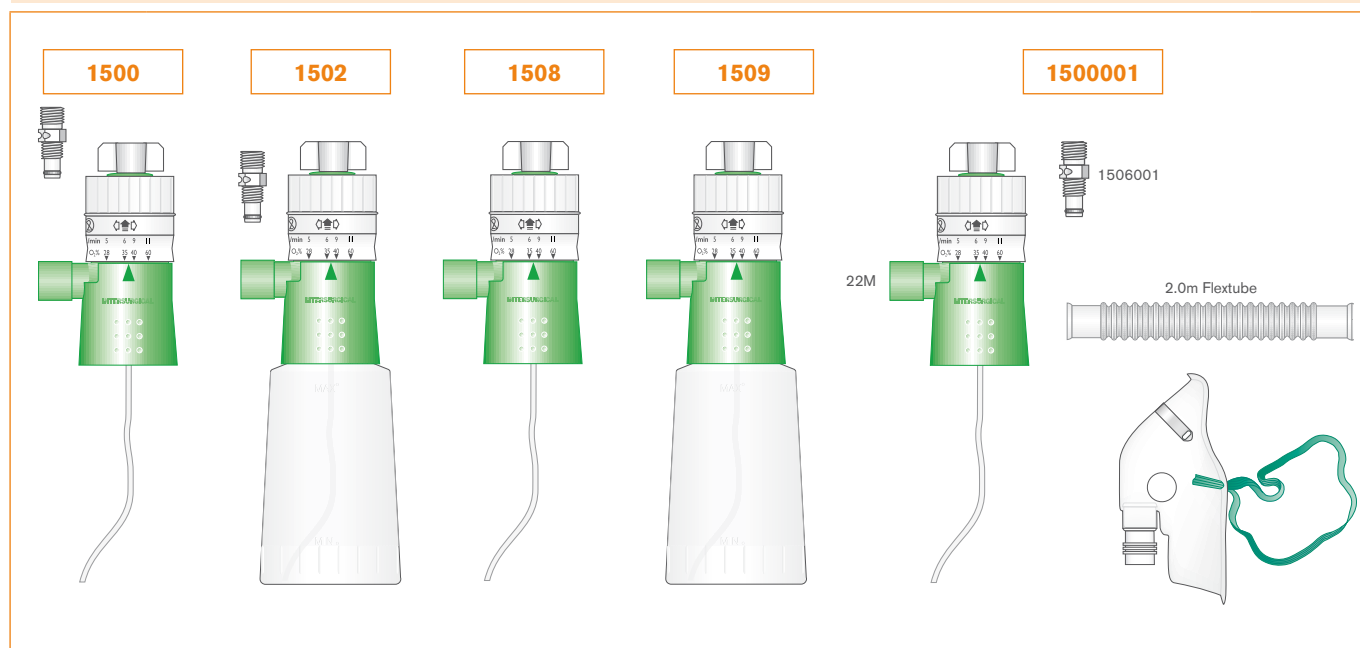

<b>1500</b>	Humidifier nebuliser + BOC adaptor without bottle	24	
<b>1502</b>	Humidifier nebuliser + BOC adaptor with bottle	25	
<b>1508</b>	Humidifier nebuliser without bottle	24	
<b>1509</b>	Humidifier nebuliser with bottle	25	
<b>150001</b>	Humidifier nebuliser + adult aerosol mask kit without bottle + BOC adaptor + 2.0m Flextube	15	

## Adaptors

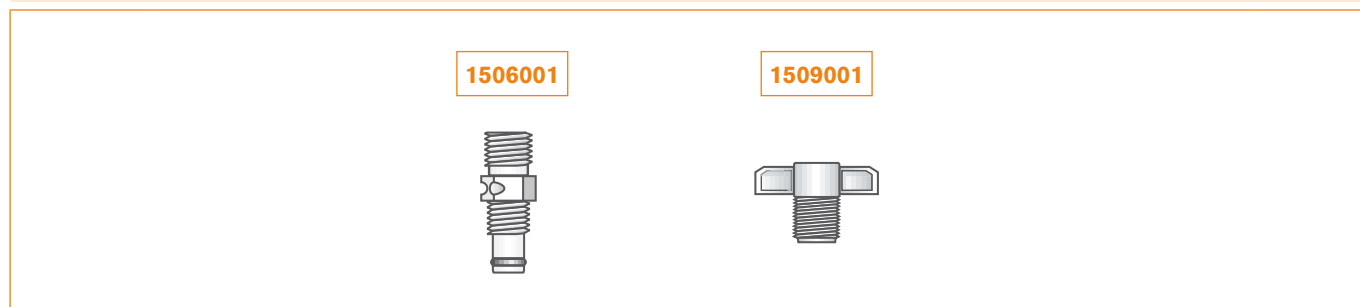

<b>1506001</b>	Flowmeter adaptor BOC	50	
<b>1509001</b>	Flowmeter adaptor M12	50	